**EN**

**Product description**

IQ Ultra™ includes 100 Patch Test Units. IQ Ultra™ is intended to be used with topical hapteners for the aid of diagnosis of contact allergy.

IQ Ultra™ consists of medical grade acrylic adhesive on a non-stick carrier tape with 10 chambers made of polyethylene foam coated with medical grade acrylic adhesive with integrated filter paper made of absorbent cellulose paper. The unit is protected by a thin layer of polyethylene. The unit is covered with a plastic cover of polystyrene coated with a thin layer of polyethylene.

**Indications for use:** IQ Ultra™ Patch Test Units are intended for use in epicutaneous diagnostic patch testing and clinical research by or under the supervision of a specially trained healthcare professional only. Patch testing is indicated in all cases where contact allergy is suspected to need to be ruled out.

**Contraindications:** None known.

**Warnings and Precautions:**

- Apply patch test units on healthy, dry and clean skin.
- Advise patients to refrain from showering, exercise or excessive sweating.
- Advise patients to protect the Patch Test Units from light (UV radiation) during ongoing patch testing.
- Wiping the test site with ethanol before application may improve adhesion of the Patch Test Unit to the patient's skin.
- Do not use adhesive reinforcing tape might be required. Consider covering shaving the hair on the test site before patch test application.
- Place the Patch Test Unit on the upper part of the patient's back if possible, avoiding the chest and upper limbs.
- Units to the midline and scapula. Avoid placing Patch Test Units under a bra strap or shoulder strap which can cause irritation to the test units.
- Severe adverse events should be reported.

**Special Patient Populations:** IQ Ultra™ Patch Test Units are used in the following forms:

- Limitations associated with the use of individual units and patch testing in general.
- Adverse events:
- Allergic reactions to tape adhesives are very rare. However, occasional cases of allergic reactions have been reported.
- In-rare reactions caused by the tape adhesive and foam frame adhesive may occur, but usually disappear rapidly.
- Stimulus may lead to the removal of the test with a slight mechanical irritation, indicated by erythema.
- Itching and burning may occur at the site of application of the Patch Test Unit. Remove the plastic cover of a positive patch test reaction or due to tape irritation.

**Instructions for Use (FIC Unit):**

- Insert IQ Ultra™ Patch Test Unit into the Application Device.
- Do not touch the adhesive surface.
- Break the perforation of a preferred corner and pull the cover downwards and upwards.
- Apply the Patch Test Unit to the patient's back or buttock by the top edge and pull the cover downwards.

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